

REPROCESSING STEPS

2

Loading
the sterilizer



2

Loading the sterilizer

2.1 START UP & PREHEATING

- Click on the display to get out of Stand-by mode.
- Close the door to speed up preheating process.



2

Loading the sterilizer

2.1 START UP & PREHEATING

- Click on the display to get out of Stand-by mode.
- Close the door to speed up preheating process.



2

Loading the sterilizer

2.2 POUCHES LOADING

- Use the trays to place the pouches.
- Avoid folds and wrinkles.
- Plastic side facing TYVEK® side.
- Place pouches in vertical position.
- Do not pile up pouches.



●●● matachana



2

Loading the sterilizer

2.2 POUCHES LOADING

- Use the trays to place the pouches.
- Avoid folds and wrinkles.
- Plastic side facing TYVEK® side.
- Place pouches in vertical position.
- Do not pile up pouches.



●●● matachana



2

Loading the sterilizer

2.2 POUCHES LOADING

- Use the trays to place the pouches.
- Avoid folds and wrinkles.
- Plastic side facing TYVEK® side.
- Place pouches in vertical position.
- Do not pile up pouches.



2

Loading the sterilizer

2.2 POUCHES LOADING

- Use the trays to place the pouches.
- Avoid folds and wrinkles.
- Plastic side facing TYVEK® side.
- Place pouches in vertical position.
- Do not pile up pouches.



2

Loading the sterilizer

2.2 POUCHES LOADING

- Use the trays to place the pouches.
- Avoid folds and wrinkles.
- Plastic side facing TYVEK® side.
- Place pouches in vertical position.
- Do not pile up pouches.



●●● matachana

NOT PILED UP



PILED UP



2

Loading the sterilizer

2.2 POUCHES LOADING

- Pull out the shelf for more comfort.
- Pouches must not touch the chamber wall.
- Keep the H₂O₂ outlet free.
- Pouches must not block the vacuum inlet.



2

Loading the sterilizer

2.2 POUCHES LOADING

- Pull out the shelf for more comfort.
- Pouches must not touch the chamber wall.
- Keep the H₂O₂ outlet free.
- Pouches must not block the vacuum inlet.



2

Loading the sterilizer

2.2 POUCHES LOADING

- Pull out the shelf for more comfort.
- Pouches must not touch the chamber wall.
- Keep the H₂O₂ outlet free.
- Pouches must not block the vacuum inlet.

●●● matachana



2

Loading the sterilizer

2.2 POUCHES LOADING

- Pull out the shelf for more comfort.
- Pouches must not touch the chamber wall.
- Keep the H₂O₂ outlet free.
- Pouches must not block the vacuum inlet.

●●● matachana



2

Loading the sterilizer

2.3 PACKS LOADING

- Small packs in the upper position.
- Bulky packs in the lower position.



●●● matachana



2

Loading the sterilizer

2.3 PACKS LOADING

- Small packs in the upper position.
- Bulky packs in the lower position.

000 matachana



000

2

Loading the sterilizer

2.4 MIXED LOAD

- When pouches are not in the basket, do not pile them.
- Bulky packs placed in the lower shelf



●●● matachana

NOT PILED UP



PILED UP



2

Loading the sterilizer

2.4 MIXED LOAD

- When pouches are not in the basket, do not pile them.
- Bulky packs placed in the lower shelf



●●● matachana

LOWER



UPPER



2

Loading the sterilizer

2.5 BIG PACKS

- Remove the upper shelfe to gain space.
- Do not pile packs.



●●● matachana



2


Loading the sterilizer

2.5 BIG PACKS

- Remove the upper shelfe to gain space.
- Do not pile packs.



NOT PILED UP

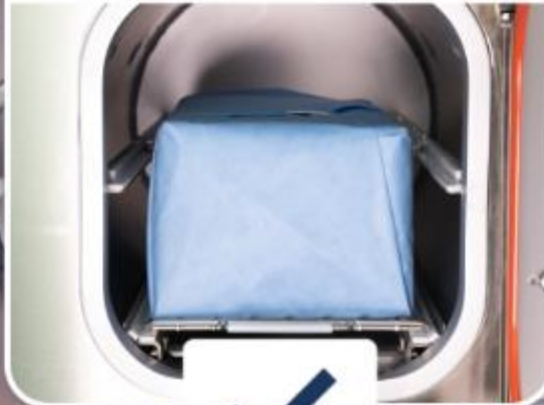

PILED UP



2

Loading the sterilizer

2.6 CLOSE THE DOOR

- Ensure the load is not blocking the door.
- Automatic door locking system. Do not push the door.



2

Loading the sterilizer

2.6 CLOSE THE DOOR

- Ensure the load is not blocking the door.
- Automatic door locking system. Do not push the door.



matachana

Home icon | Tca 53.2 °C | Pca 1011.6 hPa | Info icon

Scissors icon | Standard | Door icon 1

Cycle # 0000 | Time remaining 00:55

Play button icon

© **ANTONIO MATACHANA, S.A.** 2019

COPYRIGHT

All rights reserved. This content is the intellectual property of Antonio Matachana, S.A. Total or partial reproduction of the information is strictly prohibited, as well as its translation, copy or manipulation by any means or in any media format without the express authorization of the owner of the rights. Unauthorized use, and any breach of intellectual or industrial property rights of Antonio Matachana, S.A., will result in legal action.

CONTENT

This is a promotional video. Due to the constant evolution of our equipment, contents herein may undergo some changes.