

REPROCESSING STEPS

4

Releasing
the load




4

Releasing the load

4.1 CONFIRMING PARAMETRIC RELEASE

Confirm the cycle has been completed successfully without errors or alarms.

- No error codes or alarms in the display.
- No error codes or alarms in the printout.

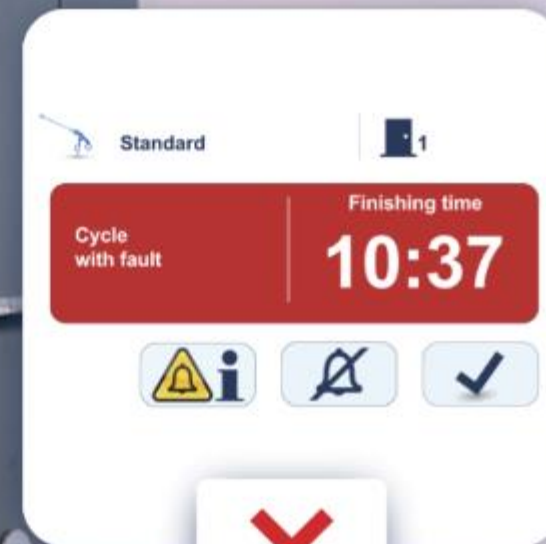

4

Releasing the load

4.1 CONFIRMING PARAMETRIC RELEASE

Confirm the cycle has been completed successfully without errors or alarms.

- No error codes or alarms in the display.
- No error codes or alarms in the printout.

Start: 08-01-2020 09:55:01
Cycle duration: 00:40:24
Ster. Duration: 00:24:24

NO INCIDENTS

00:00:00 T 55.5°C P 1028.0hPa
00:40:25 T 55.6°C P 1027.8hPa
CYCLE END



Start: 09-01-2020 15:55:12
Cycle duration: 00:17:23
Ster. Duration: 00:06:19

WITH INCIDENTS

00:00:00 T 54.2°C P 1023.1hPa
CYCLE END
00:24:15 T 55.6°C P 1021.8hPa
W48 Cycle with fault




4

Releasing the load

4.2 OPENING THE DOOR

- Click on the door icon to unlock the door.
- Door will be automatically unlocked.
- Always wear gloves and open manually the door.

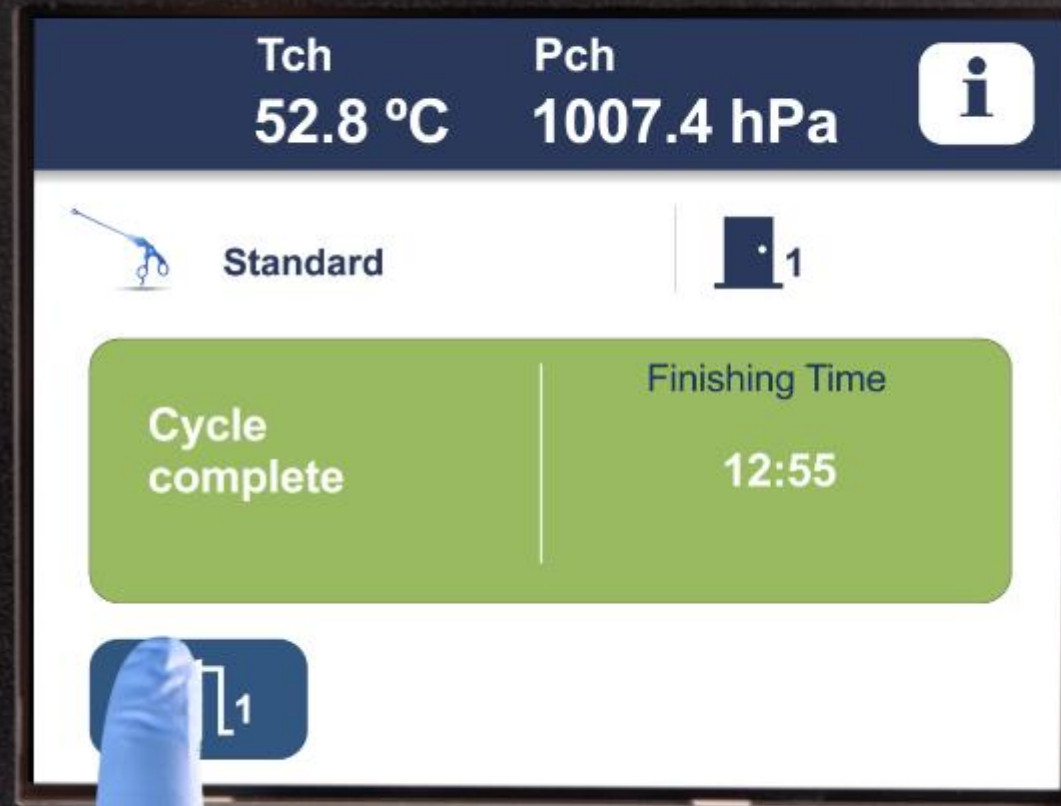

4

Releasing the load

4.2 OPENING THE DOOR

- Click on the door icon to unlock the door.
- Door will be automatically unlocked.
- Always wear gloves and open manually the door.



4

Releasing the load

4.2 OPENING THE DOOR

- Click on the door icon to unlock the door.
- Door will be automatically unlocked.
- Always wear gloves and open manually the door.



4

Releasing the load

4.3 REMOVING THE LOAD

- Pull out the shelf for more comfort.
- Place them on a table for inspection.

●●● matachana



4

Releasing the load

4.3 REMOVING THE LOAD

- Pull out the shelf for more comfort.
- Place them on a table for inspection.

●●● matachana



4

Releasing the load


4.4 CHECKING THE INDICATORS

- Confirm tape indicator.
- Confirm pouches indicator.
- Confirm the indicator inside the pouches.
- Confirm BI has been processed.
- Confirm air extraction with Helix indicator.

PROCESSED



UNPROCESSED

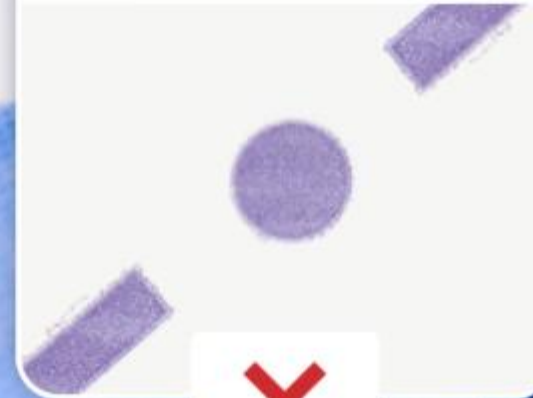

4

Releasing the load

4.4 CHECKING THE INDICATORS

- Confirm tape indicator.
- Confirm pouches indicator.
- Confirm the indicator inside the pouches.
- Confirm BI has been processed.
- Confirm air extraction with Helix indicator.

●●● matachana

PROCESSED



UNPROCESSED



4

Releasing the load

4.4 CHECKING THE INDICATORS

- Confirm tape indicator.
- Confirm pouches indicator.
- Confirm the indicator inside the pouches.
- Confirm BI has been processed.
- Confirm air extraction with Helix indicator.

●●● matachana



PROCESSED

UNPROCESSED



4

Releasing the load

4.4 CHECKING THE INDICATORS

- Confirm tape indicator.
- Confirm pouches indicator.
- Confirm the indicator inside the pouches.
- Confirm BI has been processed.
- Confirm air extraction with Helix indicator.

●●● matachana



REMEMBER TO FOLLOW INCUBATION PROCEDURE ACCORDING TO MATACHANA INSTRUCTIONS.

PROCESSED



UNPROCESSED





4

Releasing the load

4.4 CHECKING THE INDICATORS

- Confirm tape indicator.
- Confirm pouches indicator.
- Confirm the indicator inside the pouches.
- Confirm BI has been processed.
- Confirm air extraction with Helix indicator.

000 matachana

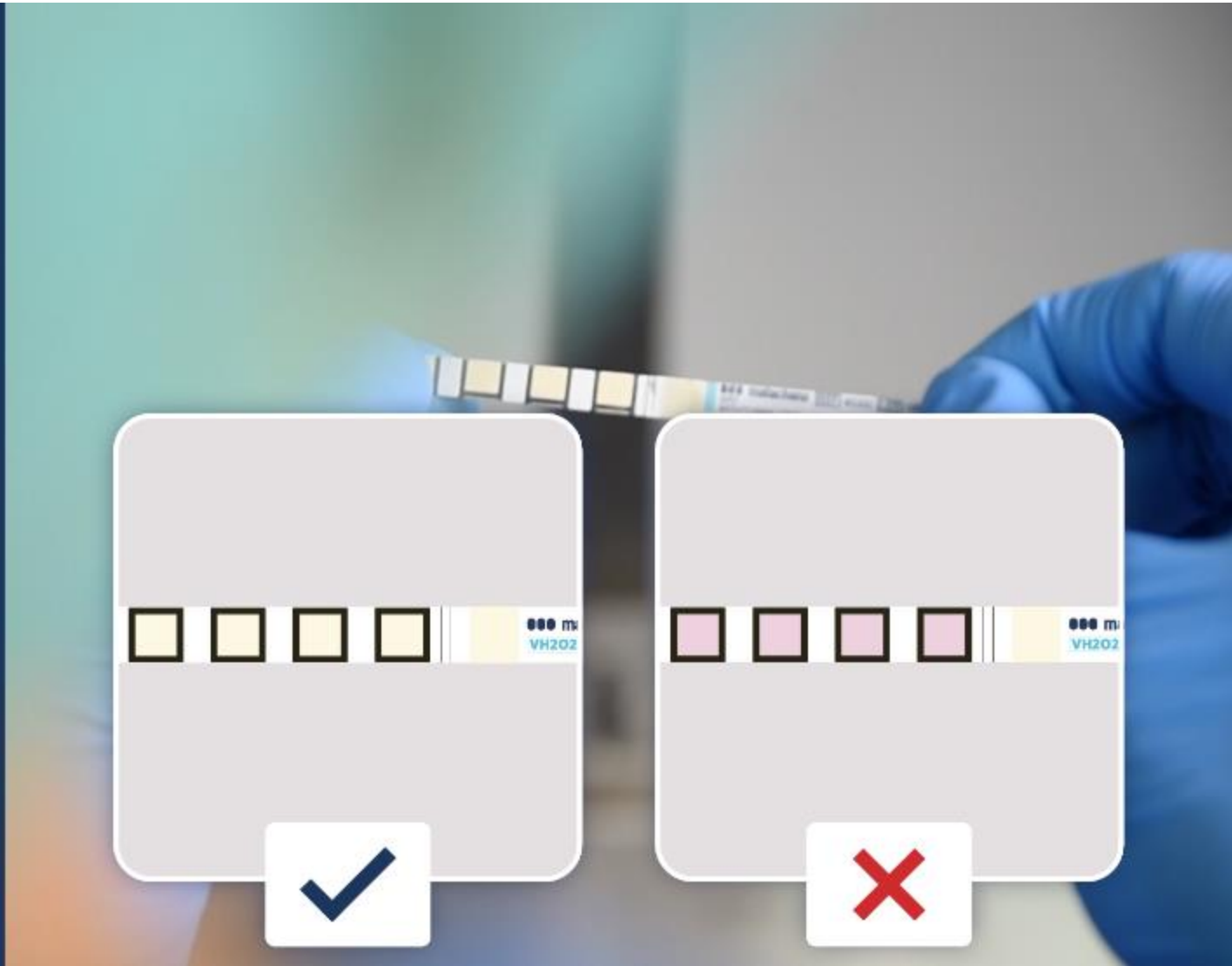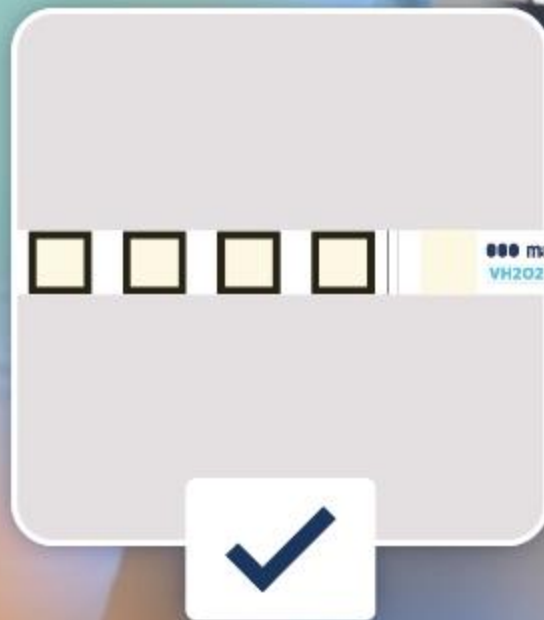

© **ANTONIO MATACHANA, S.A.** 2019

COPYRIGHT

All rights reserved. This content is the intellectual property of Antonio Matachana, S.A. Total or partial reproduction of the information is strictly prohibited, as well as its translation, copy or manipulation by any means or in any media format without the express authorization of the owner of the rights. Unauthorized use, and any breach of intellectual or industrial property rights of Antonio Matachana, S.A., will result in legal action.

CONTENT

This is a promotional video. Due to the constant evolution of our equipment, contents herein may undergo some changes.