

BLUKAT® REPLACEMENT





BluKat[®] Replacement

1. CHECKING THE BLUKAT[®] LEVEL

- Verify Led Color to know the sterilizing solution level.
- Click on the BluKat[®] icon to know sterilizing solution level details.



130HPO[®]



18
Cycles

50HPO[®]



30
Cycles



130HPO[®]



5
Cycles

50HPO[®]



8
Cycles



130HPO[®]



2
Cycles

50HPO[®]



2
Cycles



**BluKat®
Replacement**

1. CHECKING
THE BLUKAT® LEVEL

- Verify Led Color to know the sterilizing solution level.
- Click on the BluKat® icon to know sterilizing solution level details.

Tch
52.8 °C

Pch
1007.4 hPa



BluKat:  30 cycles



Rapid



Advanced



Standard





BluKat[®] Replacement

1. CHECKING THE BLUKAT[®] LEVEL

- BluKat[®] Batch no. and expiration date.
- Remaining no. of cycles.
- Remaining days before expiration.



Tch
52.8 °C

Pch
1007.4 hPa



BluKat



Batch Number: 18001
Expiry date: 07-2021
Remaining: 30 cycles Valid: 30 days





BluKat® Replacement

1. CHECKING THE BLUKAT® LEVEL

- BluKat® Batch no. and expiration date.
- Remaining no. of cycles.
- Remaining days before expiration.

●●● **matachana**



Tch
52.8 °C

Pch
1007.4 hPa



BluKat



Batch Number:

18001

Expiry date:

07-2021

Remaining: 30 cycles

Valid: 30 days





BluKat[®] Replacement

1. CHECKING THE BLUKAT[®] LEVEL

- BluKat[®] Batch no. and expiration date.
- Remaining no. of cycles.
- Remaining days before expiration.

●●● matachana



Tch
52.8 °C

Pch
1007.4 hPa



BluKat



Batch Number:


18001

Expiry date:

07-2021

Remaining: 30 cycles

Valid: 30 days

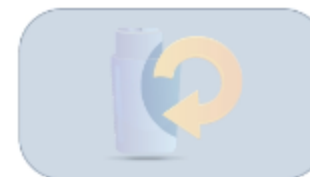



BluKat[®] Replacement

2. PERFORMING THE EMPTYING CYCLE

- Sterilizer will indicate when there is no more sterilizing solution or if expired.
- Press emptying cycle button.
- Press the "play" button.

When there is no more
sterilization solution available



Tch
52.8 °C

Pch
1007.4 hPa



PROCESS STOPPED

**E21 Sterilization solution
depleted. Replace BluKat**





BluKat[®] Replacement

2. PERFORMING THE EMPTYING CYCLE

- Sterilizer will indicate when there is no more sterilizing solution or if expired.
- Press emptying cycle button.
- Press the "play" button.

When the sterilization solution is expired.



Tch	Pch
52.8 °C	1007.4 hPa

 PROCESS STOPPED

E20 BluKat expired




BluKat® Replacement

2. PERFORMING THE EMPTYING CYCLE

- Sterilizer will indicate when there is no more sterilizing solution or if expired.
- Press emptying cycle button.
- Press the "play" button.

When the BluKat® cartridge needs to be removed.

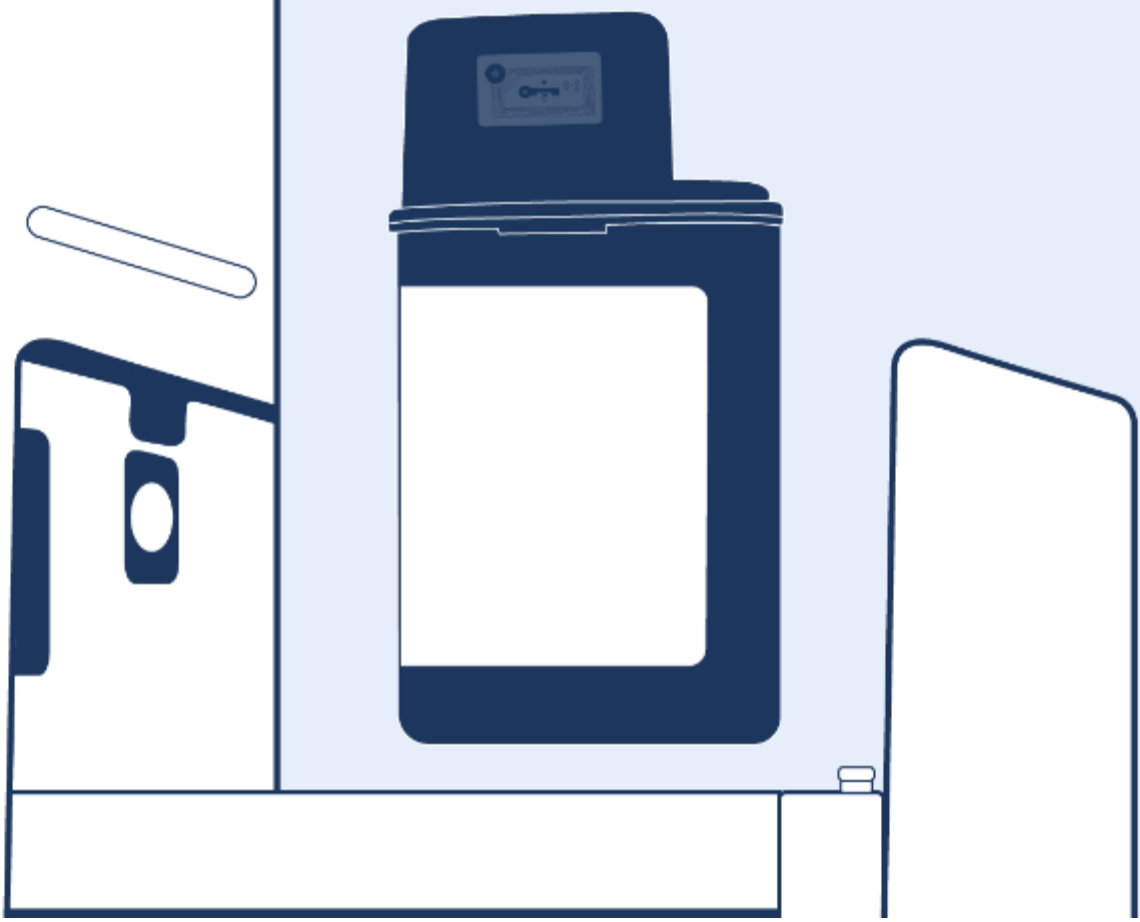



BluKat® Replacement

2. PERFORMING THE EMPTYING CYCLE

- Sterilizer will indicate when there is no more sterilizing solution or if expired.
- Press emptying cycle button.
- Press the "play" button.

●●● matachana


HPO

Tch
52.8 °C

Pch
1007.4 hPa




BluKat  0 cycles

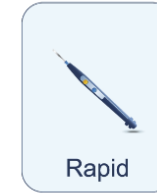

Rapid



Advanced

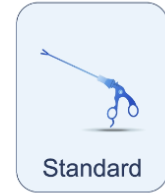

Standard





BluKat[®] Replacement

2. PERFORMING THE EMPTYING CYCLE

- Sterilizer will indicate when there is no more sterilizing solution or if expired.
- Press emptying cycle button.
- Press the "play" button.

●●● matachana



Tch
52.8 °C

Pch
1007.4 hPa



BluKat



Batch Number:

18001

Expiry date:

07-2021

Remaining: **0 cycles**

Valid: 8 days





**BluKat®
Replacement**

2. PERFORMING THE
EMPTYING CYCLE

- Sterilizer will indicate when there is no more sterilizing solution or if expired.
- Press emptying cycle button.
- Press the “play” button.

●●● **matachana**



**Tch
52.8 °C**

**Pch
1007.4 hPa**



**BluKat
disposal**



Remaining time

00:33





BluKat®
Replacement

2. PERFORMING THE
EMPTYING CYCLE

- Confirm BluKat® emptying cycle.
- Total time will depend on the amount of remaining sterilizing solution.

●●● matachana

Tch
52.8 °C

Pch
1007.4 hPa



This is a **TEST PROGRAM**
The material will not be sterile!
Run this test?





BluKat®
Replacement

2. PERFORMING THE
EMPTYING CYCLE

- Confirm BluKat® emptying cycle.
- Total time will depend on the amount of remaining sterilizing solution.

●●● matachana



Tch
52.8 °C

Pch
1007.4 hPa



BluKat
disposal



PROCESS
RUNNING

Remaining time

00:25





BluKat® Replacement

3. REMOVING BLUKAT® FROM THE STERILIZER

- Ensure you wear gloves.
- Confirm the emptying cycle has ended by pressing the icon.





BluKat® Replacement

3. REMOVING BLUKAT® FROM THE STERILIZER

- Ensure you wear gloves.
- Confirm the emptying cycle has ended by pressing the icon.



Tch
52.8 °C

Pch
1007.4 hPa



Finished
BluKat



Complete

Replace
BluKat!





BluKat® Replacement

3. REMOVING BLUKAT® FROM THE STERILIZER

- Pull the door to have access to the BluKat® cartridge.
- Pull out the cartridge.
- Follow local regulations for discarding empty cartridge.





BluKat® Replacement

3. REMOVING BLUKAT® FROM THE STERILIZER

- Pull the door to have access to the BluKat® cartridge.
- Pull out the cartridge.
- Follow local regulations for discarding empty cartridge.

●●● matachana





BluKat® Replacement

3. REMOVING BLUKAT® FROM THE STERILIZER

- Pull the door to have access to the BluKat® cartridge.
- Pull out the cartridge.
- Follow local regulations for discarding empty cartridge.

●●● matachana






BluKat® Replacement

4. REMOVING NEW BLUKAT® FROM THE BOX

- Open the box and follow the instructions.

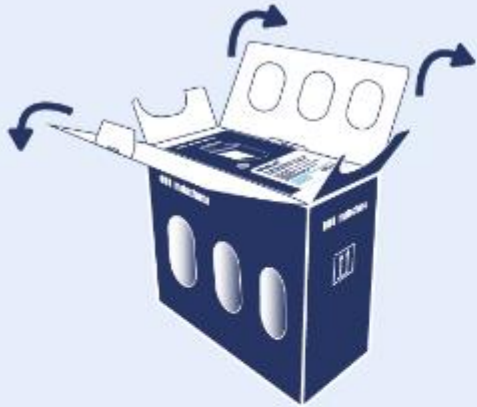

●●● matachana





BluKat® Replacement

4. REMOVING NEW BLUKAT® FROM THE BOX

- Open the box and follow the instructions.





BluKat[®] Replacement

4. REMOVING NEW BLUKAT[®] FROM THE BOX

- Open the box and follow the instructions.



●●● matachana





BluKat® Replacement

4. REMOVING NEW BLUKAT® FROM THE BOX

- Open the box and follow the instructions.



●●● matachana





BluKat® Replacement

4. REMOVING NEW BLUKAT® FROM THE BOX

- Open the box and follow the instructions.





BluKat® Replacement

5. PLACING NEW BLUKAT® INTO THE STERILIZER

- Press the icon to unlock the BluKat® compartment.
- Ensure you wear gloves.
- Introduce the BluKat® in its drawer, ensuring it is placed in the right position.



Tch
52.8 °C

Pch
1007.4 hPa



BluKat



Batch Number:

Expiry date:

Remaining: cycles Valid: days





BluKat® Replacement

5. PLACING NEW BLUKAT® INTO THE STERILIZER

- Press the icon to unlock the BluKat® compartment.
- Ensure you wear gloves.
- Introduce the BluKat® in its drawer, ensuring it is placed in the right position.

●●● matachana





BluKat® Replacement

5. PLACING NEW BLUKAT® INTO THE STERILIZER

- Press the icon to unlock the BluKat® compartment.
- Ensure you wear gloves.
- Introduce the BluKat® in its drawer, ensuring it is placed in the right position.

●●● matachana





BluKat® Replacement

5. PLACING NEW BLUKAT® INTO THE STERILIZER

- Push in the drawer to close it.
- Automatically, the sterilizer will read the rfid tag.
- New BluKat® information will be displayed.





BluKat® Replacement

5. PLACING NEW BLUKAT® INTO THE STERILIZER

- Push in the drawer to close it.
- Automatically, the sterilizer will read the rfid tag.
- New BluKat® information will be displayed.





BluKat® Replacement

5. PLACING NEW BLUKAT® INTO THE STERILIZER

- Push in the drawer to close it.
- Automatically, the sterilizer will read the rfid tag.
- New BluKat® information will be displayed.



Tch
52.8 °C

Pch
1007.4 hPa



BluKat



Batch Number: 18001
 Expiry date: 07-2021
 Remaining: 30 cycles Valid: 30 days



© **ANTONIO MATACHANA, S.A.** 2019

COPYRIGHT

All rights reserved. This content is the intellectual property of Antonio Matachana, S.A. Total or partial reproduction of the information is strictly prohibited, as well as its translation, copy or manipulation by any means or in any media format without the express authorization of the owner of the rights. Unauthorized use, and any breach of intellectual or industrial property rights of Antonio Matachana, S.A., will result in legal action.

CONTENT

This is a promotional video. Due to the constant evolution of our equipment, contents herein may undergo some changes.